Aquatec® Pico / Pico TS

- EN** Shower stool / Commode chair
User Manual
- DE** Duschhocker / Toilettenstuhl
Gebrauchsanweisung
- FR** Tabouret et chaise de douche / Chaise percée avec accoudoirs et seau
Mode d'emploi
- IT** Sgabello per la doccia / Sedia da comodo
Istruzioni per l'uso
- ES** Taburete de ducha / Silla WC
Manual de instrucciones
- NL** Douchekruk / Toiletstoel
Gebruiksaanwijzing
- DA** Badetaburet / Toiletstol
Brugsanvisning
- FI** Suihkujakkara / WC-tuoli
Käyttöohje
- NO** Dusjkrakk / Toalettstol
Brukerveiledning
- SV** Duschpallen / Toalettstol
Bruksanvisning
- PT** Banco de duche / Cadeira de casa de banho
Manual de instruções
- ET** Duši-/tualetitool
Kasutusjuhend
- LT** Dušo / tualetu kėdutė
Veikimo nurodymai
- LV** Dušas krēsls / tualetes krēsls
Lietošanas instrukcija

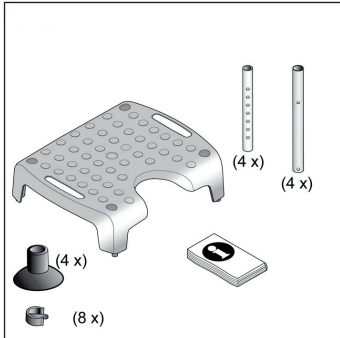
Fig. 1

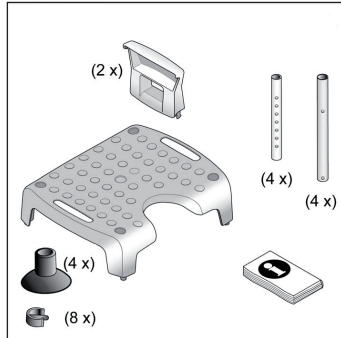
Fig. 2

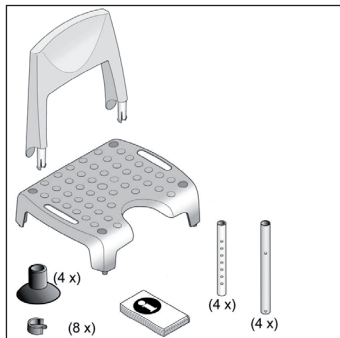
Fig. 3

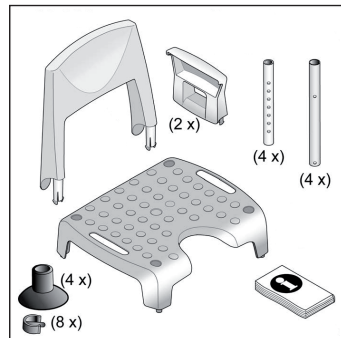
Fig. 4

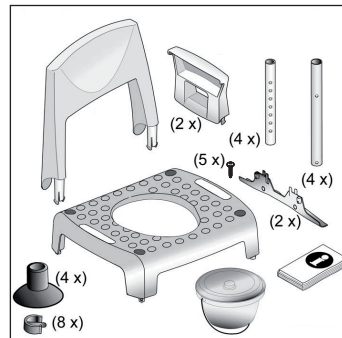
Fig. 5

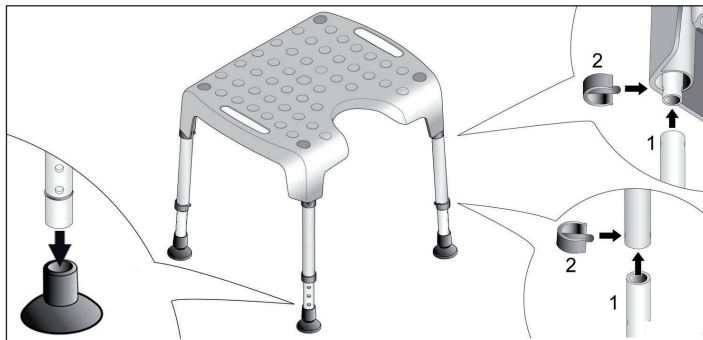
Fig. 6

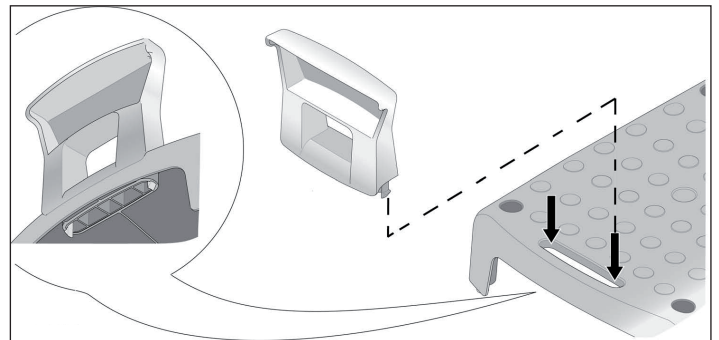
Fig. 7

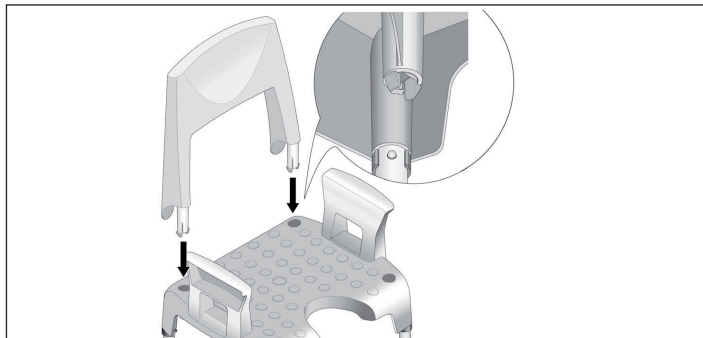
Fig. 8

AQUATEC PICO komplett
with arm- and backrest
2012-05 160 kg 40286984033804
Made in Germany [LN] (01) 04028698083904 / (10) 12EDA0001

AQUATEC PICO TS
2012-05 160 kg 40286984111699
Made in Germany [LN] (01) 04028698111699 / (10) 12EDA0001

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can!

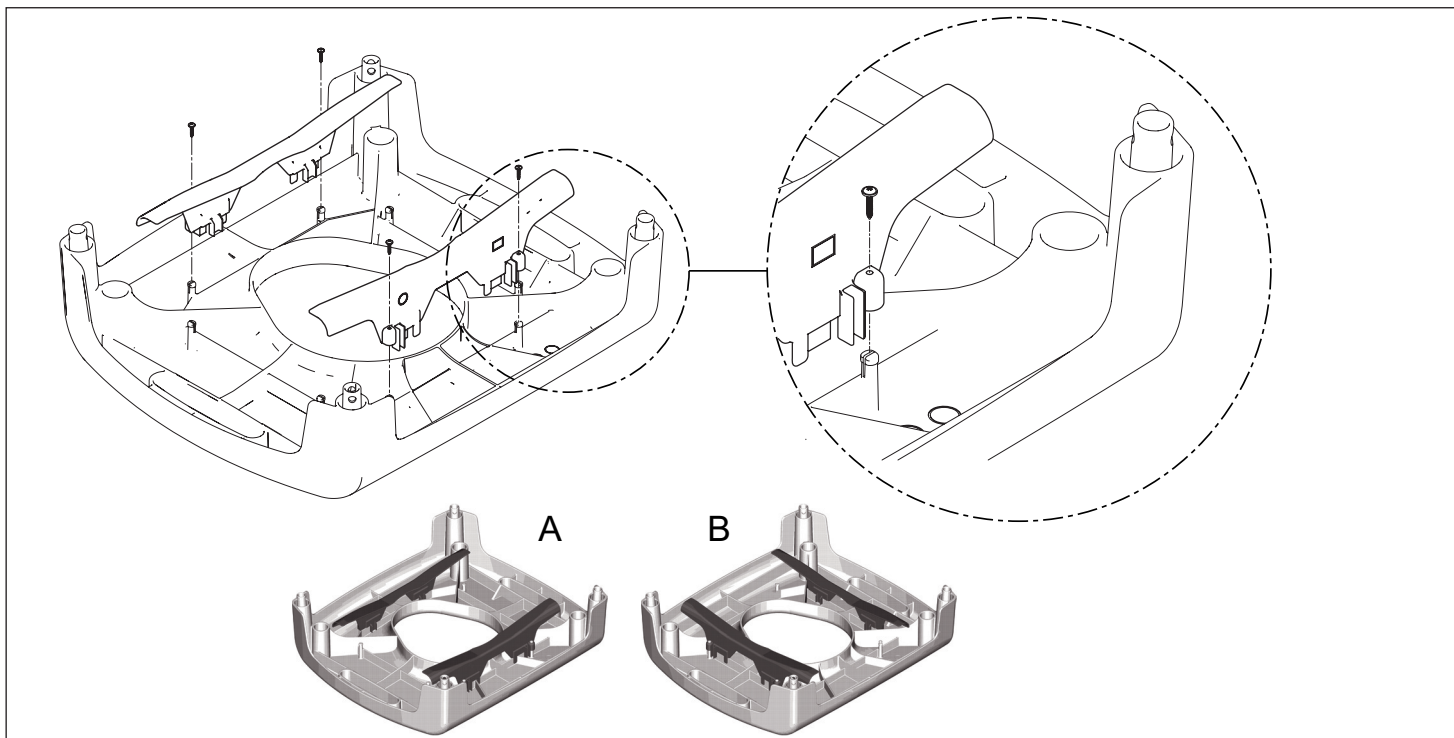
Fig. 9

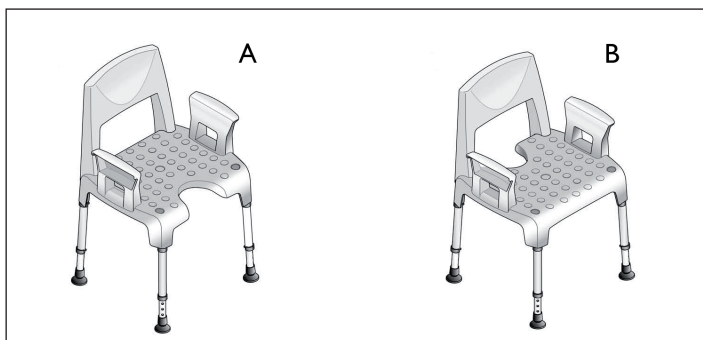
Fig. 10

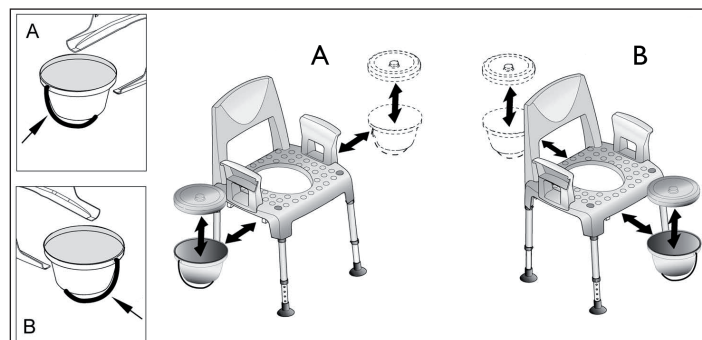
Fig. 11

Symbols in this user manual

EN

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer

Intended Use

The Shower stool / Commode chair is to be used solely as an aid for showering or going to the toilet. Any other use is prohibited.

Temperature for use: 10 - 40 °C

Max. user weight: 160 kg

Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

WARNING Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

Scope of delivery

- Fig. 1 - 5. (Depends on model)

In the scope of delivery (Fig. 5) there are five screws included for assembling the two toilet pan holders. Only four screws are required for assembly.

Assembly

CAUTION Risk of pinching!

- Take care during assembly.

WARNING Risk of falling!

- Ensure that all legs are adjusted to the same height.

- Fig. 6 - 9.

Usage

- Fig. 10, 11.

Cleaning and disinfection

IMPORTANT

- Take off any fitted accessories.

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

! IMPORTANT

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 85 °C for 3 minutes.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

! WARNING

- Check product regularly for damage and to ensure that it is securely assembled.
- Check the arm and backrest fixings and the clips regularly to ensure they are sitting securely.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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
Technical Data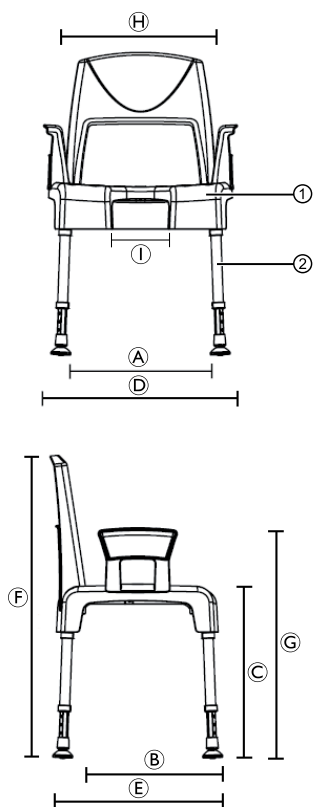

Order numbers	Aquatec® Pico
Shower stool	9.20.001
Shower stool with armrests	9.21.001
Shower chair with backrest	9.23.001
Shower chair complete	9.22.001
Backrest	16469
Armrests (pair)	16468
Measurements	
A	430 mm
B	420 mm
C	425 - 575 mm
D	575 mm
E	520 mm
F	825 - 975 mm
G	605 - 755 mm
H	430 mm
I	180 mm
Depth, hygiene recess	145 mm
Graduation, height adjustment	25 mm

Weights	
Shower stool	3.2 kg
Shower stool with armrests	4.1 kg
Shower chair with backrest	4.0 kg
Shower chair complete	4.8 kg
Max. User Weight	160 kg
Color	White
Materials	
①	PP
②	Aluminium, corrosion-free

Invacare® Distributors:

Belgium & Luxembourg:

Invacare nv, Autobaan 22,
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Deutschland:

Invacare GmbH, Alemannenstrasse 10,
D-88316 Isny
Tel: (49) (0)75 62 7 00 0
Fax: (49) (0)75 62 7 00 66
kontakt@invacare.com / www.invacare.de

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101,
A-5310 Mondsee
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Schweiz:

Invacare AG, Benkenstrasse 260,
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62,
I-36016 Thiene
Tel: (39) 0445 38 00 59
Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46,
NL-6716 BZ Ede
Tel: (31) (0)318 695 757
Fax: (31) (0)318 695 758
nederland@invacare.com / www.invacare.nl

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Cam-
pus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed,
Bridgend CF35 5AQ
Tel: (44) (0)1656 776200
Fax: (44) (0)1656 776201
UK@invacare.com / www.invacare.co.uk

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà,
E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00
Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch,
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactcfr@invacare.com / www.invacare.fr

Denmark:

Invacare A/S, Sdr. Ringvej 37,
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230,
Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00
Fax: (47) (0)22 57 95 01
norway@invacare.com / www.invacare.no
island@invacare.com

Sverige:

Invacare AB, Fagerstagatan 9,
S-163 91 Spånga
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com / www.invacare.se

Suomi:

Algol-Trehab Oy
Karapellontie 6, 02610 Espoo
Puhelin 09 50991
Faksi 09 5099490
Sähköposti info.algoltrehab.fi@algol.fi
Internet www.algoltrehab.fi

Portugal:

Invacare Lda, Rua Estrada Velha, 949,
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

USA:

CLARKE HEALTH CARE PRODUCTS INC.
1003 International Dr. USA, Oakdale, PA 150719226
Tel: 0724 69 52 122
Fax: 0724 69 52 922
info@clarkehealthcare.com / www.clarkehealthcare.com

Canada:

INVACARE® CANADA INC., 570 Matheson Blvd. East,
Unit 8, CDN Mississauga, On. L4Z4G4
Tel. 0905 8908300
Fax. 0905 8905244
www.invacare.ca

Eastern Europe, Middle East, Russia & CIS:

European Distributor Organisation:
Invacare, Kleiststraße 49
D-32457 Porta Westfalica
Tel. (49) (0)57 31 754 540
Fax. (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

Aquatec Operations GmbH
Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0
Fax: (49) (0)75 62 7 00 66
kontakt@invacare.com / www.invacare.de

AQUATEC

INVACARE